



EasyBloodGas™

easy inside and out

With EasyBloodGas, Medica has redefined blood gas analyzer design

The sophistication of traditional blood gas analyzers has been packaged in a new compact format with a small footprint to save space. Gas tanks are eliminated. Gas and liquids have been mixed (tonometered) to create a convenient Reagent Module containing liquid calibrants. All components are combined into three simple modules, easily accessible by the user. Routine maintenance is limited to the replacement of electrodes and a single pump tube.

Medica's EasyBloodGas analyzer measures pH, PCO_2 and PO_2 and calculates eleven additional parameters. Patient parameters, including FIO_2 and Hb, can be entered using the digital keypad and integrated into patient results. Measured and calculated results are displayed and printed. Simple menus guide the user through analyzer operation. The modular design makes maintenance easy.


EasyBloodGas focuses on the laboratory's need to deliver sample results economically. Unique electrode design, combined with precise control of calibrator volumes, ensure economical operation and a low cost per sample.



... Easy to use

Blood gas analyzer operation has never been simpler

The Universal Sampler adapts to both syringe and capillary samples. The sample probe's self-wiping feature provides convenience, sample integrity and user safety.

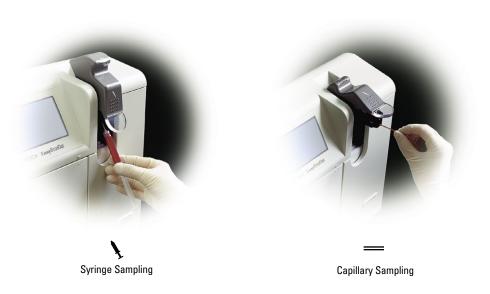

Compact reagent module for convenience, economy and safety

Bulky gas tanks are replaced with liquid, tonometered calibrants, packaged in a convenient Reagent Module which also collects waste, protecting the user from biological hazards. The Reagent Module's solid-state memory enables EasyBloodGas to track date code and reagent usage. Operation without interruption is assured.

Disposable, maintenance-free electrodes

PH PCO, PO,

Advanced membrane technology and novel packaging bring unprecedented convenience to electrode replacement. Medica's integral membrane design means that membranes never need to be changed by the user, saving time and simplifying maintenance.

HOME MENU

- ANALYZE SAMPLE ANALYZE QC 1 2 3
- CALIBRATE DAILY CLEANER 4
- 5 SECOND MENU

HD ME

SETUP MENU

- SECURITY CODE 1
- USER OPTIONS CONFIGURATION 2 3
 - REFERENCE LIMITS
- 4 5 PATIENT INFORMATION DELETE DATA
- 6

DIAGNOSTICS

- TEST COMPONENTS 1
- TEST FLUIDICS SENSOR STATUS 2 3
- PRIME FLUIDS 4

Simple menu-prompted operation with a touch of the keypad

002

2CO2

EasyBloodGas can be programmed to conform with established lab protocols. The software allows selection of desired options, including Reference Limits, Quality Control Limits, Operator ID and Patient Data.

Sensor Module contains Universal Sampler with self-wiping probe for convenience and safety

Valve Module contains long-lasting ceramic valve; selects and preheats reagents Maintenance-free Electrode design permits fast, fail-safe installation

... Easy to maintain

EasyBloodGas can be maintained by anyone, anytime, anywhere

Innovative design simplifies maintenance, addressing the needs of the remote laboratory with limited access to technical service personnel. All service calls can be performed by fax or telephone, eliminating the need for on-site service. Diagnostic software displays component status, assuring quick troubleshooting. Modularity makes assembly and disassembly quick and easy.

Removal of the three plug-in modules—Reagent Module, Sensor Module and Valve Module—is accomplished without tools.

... Easy data management

Comprehensive quality control and data management

The EasyBloodGas quality control program calculates and stores complete statistics for the last 30 quality control results at each of three levels. A printed Levey-Jennings chart visually identifies trends. The data management program compares all patient results with ranges stored in memory and flags out-of-range results. Results are stored in memory for up to 64 patients.



Ref

Self-contained **Reagent Module** eliminates gas tanks and collects all waste

Specifications

CLIA Classification: Sample Type: Sample Size:

Measured Parameters

рΗ PCO₂ PO_2

Calculated Parameters

pH (T) (pH temperature corrected)
----------	---------------------------

PCO₂ (T) (PCO₂ temperature corrected) PO₂ (T) (PO₂ temperature corrected)

- TCO₂ (Total Carbon Dioxide)
- HCO3⁻ (Bicarbonate)
- BE_b (Base Excess in blood)
- BE_{ecf} (Base
- SBC (Standa
- %SO₂C (Oxy
- A-aDO₂ (Alve
- RI (Respirato

Input Param

Patient Tempe Hemoglobin FIO₂ Patient ID

Sample Tem

Ambient Co

Analysis Tim

Data Storag


Calibration:

Input/Output

Power:

Size & Weig

Moderate complexity Whole blood 100 μ L Syringe 75 μ L Capillary

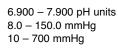

0 - 50 mmol/L

0 - 50 mmol/L

-25.0 - 25.0 mmol/L



Medica Corporation 5 Oak Park Drive Bedford, MA 01730-1413 USA

USA 800 777 5983

Global +1 781 275 4892

Fax 781 275 2731

medicacorp.com

Excess in extra cellular fluid)	-25.0 – 25.0 mmol/L
lard Bicarbonate)	0 – 50 mmol/L
ygen Saturation calculated at normal P_{50})	40.0 - 100.0%
veolar arterial oxygen gradient)	0 – 700 mmHg
tory Index)	0.0 – 70.0
neters perature	5 – 45°C 30 – 30.0 g/dL 10 – 100% 14 digits
nperature Control:	37°C ± 0.2°C
onditions:	15 – 30°C (59 – 86°F), 500 – 800 mmHg (max 15 PSI) 5 – 85% relative humidity, non-condensing atmospheric air environment (21% PO ₂)
me:	125 seconds
ge:	64 Patient results with Operator ID, Patient ID, Date and Time $QC - up$ to 30 results for each Level (1, 2, 3)
:	Automatic or On-Demand
ut:	Numeric keypad, graphic display, 27 column thermal printer, barcode reader port, RS-232 computer interface port
	100/115 ~ VAC, 50 – 60 Hz, 0.8 A or 220 ~ VAC, 50 – 60 Hz, 0.4A
ght:	14.5" W x 12.5" H x 7" D (37 cm W x 32 cm H x 18 cm D) 16 lbs. (7.3 kg) with reagent module